

Sending Your Sample

Kits should be sent on the same day of collection

1. Place the blood collection tube into the **larger transport tube. Put cap securely into place**
2. If not already done, write patient name and date on the label and stick it on the transport tube.
3. Place **the transportation tube, disposal bag** with lancet and test order form back into the bag.
4. Place **completed requisition in the bag along with blood sample and disposal bag with used lancet.**
5. Place the bag into **Courier pack.**
6. **Complete the pre-printed Courier waybill** with your name and address.
7. **DO NOT** ship on Fridays, or before Holiday Seasons and Keep Refrigerated until shipping
8. Either
 - a. Return the package to your healthcare provider for shipment to the lab. **OR**
 - b. Drop the parcel with the way bill off at the Post office.

AVAZERA

FOOD INTOLERANCE BLOOD COLLECTION INSTRUCTIONS

Thank you for choosing the CanAlt Food IgG intolerance test. This kit contains all the necessary materials to collect and ship the sample. You should receive your results within 7 – 10 days from the laboratory receiving the sample. Your report will contain a Patient Guide, which helps explain your results. Please contact your Health Care Provider if you have further questions.

Kit Contents:

- ✓ Patient Requisition Form
- ✓ Collection Instructions
- ✓ Two Safety lancets
- ✓ Alcohol Swab
- ✓ Sterile absorbent gauze
- ✓ Band-Aid
- ✓ Blood collection tube (**Check Expiry date**)
- ✓ Larger plastic tube with label for transportation
- ✓ Disposal bag
- ✓ Courier shipping pack and pre-printed waybill
- ✓ Name writing label

